



SINGLE MWA PERCUTANEOUS ANTENNA

The following illustration and table depict the approximate size of an ablation zone that may be created by a single Evident™ MW Ablation Percutaneous Antenna energized at various power and time settings. The shaded regions in the images represent the zone of tissue coagulation created around the antenna.

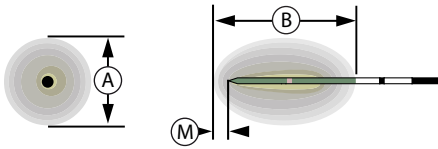

	30 W	45 W	
	10:00	5:00	10:00
A	2.2 cm	2.3 cm	2.8 cm
B	3.3 cm	3.2 cm	4.0 cm
M	0.5 cm	0.6 cm	0.7 cm
	9 cm ³	9 cm ³	16 cm ³