


THREE MWA PERCUTANEOUS ANTENNAS

The following illustration and table depict the approximate size of an ablation zone that may be created by three Evident™ MW Ablation Percutaneous Antennas energized simultaneously at various power and time settings using various spacings.

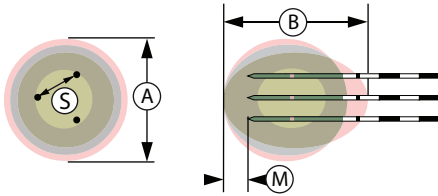

	30 W	10:00	45 W	5:00	45 W	10:00
S	1.5 cm	2.0 cm	1.5 cm	2.0 cm	1.5 cm	2.0 cm
A	4.2 cm	4.3 cm	3.6 cm	3.7 cm	4.2 cm	4.6 cm
B	4.4 cm	4.2 cm	4.3 cm	4.2 cm	5.6 cm	5.6 cm
M	0.7 cm	0.7 cm	0.6 cm	0.6 cm	0.8 cm	0.8 cm
	40 cm ³	40 cm ³	30 cm ³	30 cm ³	53 cm ³	63 cm ³