


CLUSTER MWA PERCUTANEOUS ANTENNAS CONFIGURATION FOR PLANAR COAGULATION

The following illustration and table depict the approximate size of an ablation zone that may be created by multiple Evident™ MW Ablation Percutaneous Antennas in a cluster configuration of 0.5 cm between each antenna energized simultaneously at 45 W and 30 seconds. When performing planar coagulation prior to resection with multiple antennas, do not space antennas closer than 0.5 cm for ablation time greater than 30 seconds.

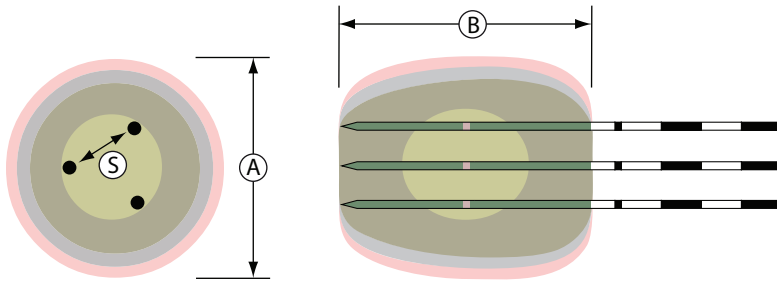

45 W ↻ 0:30

S 0.5 cm

A 1.1 cm

B 3.2 cm