


SINGLE MWA SURGICAL ANTENNA

The following illustration and table depict an ablation zone that may be created by a single Evident™ MW Ablation Surgical Antenna at various power and time settings. The shaded regions in the images represent the zone of tissue coagulation created around the antenna.

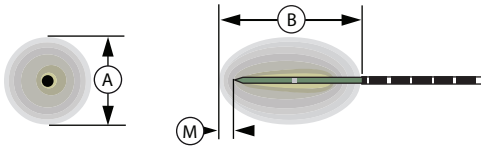

	30 W		45 W	
	10:00	5:00	10:00	
A	2.6 cm	2.2 cm	2.7 cm	
B	3.6 cm	3.9 cm	4.1 cm	
M	0.5 cm	0.6 cm	0.6 cm	
	12 cm ³	10 cm ³	16 cm ³	