


THREE MWA SURGICAL ANTENNAS

The following illustration and table depict the approximate size of an ablation zone that may be created by three Evident™ MW Ablation Surgical Antennas energized simultaneously at various power and time settings using various spacings.

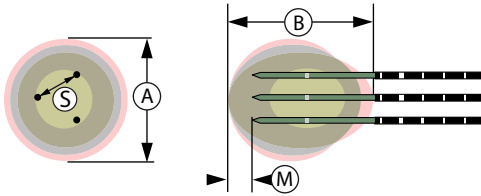

S	30 W 1.5 cm	10:00 2.0 cm	45 W 1.5 cm	5:00 2.0 cm	45 W 1.5 cm	10:00 2.0 cm
A	4.3 cm	4.4 cm	4.0 cm	4.0 cm	4.1 cm	4.6 cm
B	5.0 cm	4.6 cm	5.0 cm	4.9 cm	4.6 cm	5.1 cm
M	0.8 cm	0.8 cm	0.6 cm	0.6 cm	0.8 cm	0.8 cm
	49 cm ³	47 cm ³	42 cm ³	41 cm ³	40 cm ³	56 cm ³