





CLUSTER MWA SURGICAL ANTENNAS CONFIGURATION FOR PLANAR COAGULATION

The following illustration and table depict the approximate size of an ablation zone that may be created by multiple Evident™ MW Ablation Surgical Antennas in a cluster configuration of 0.5 cm between each antenna energized simultaneously at 45 W and 30 seconds. When performing planar coagulation prior to resection with multiple antennas, do not space antennas closer than 0.5 cm for ablation time greater than 30 seconds.

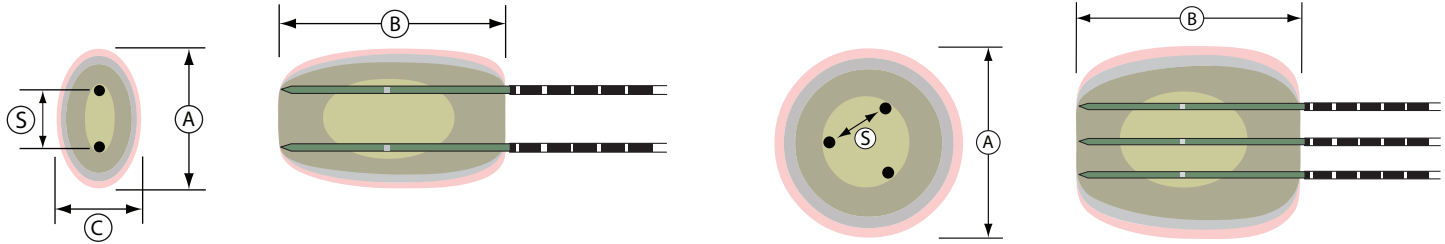

45 W  0:30		
S	0.5 cm	0.5 cm
A	1.3 cm	1.2 cm
B	3.2 cm	3.5 cm
C	1.0 cm	NA